Description

Helioseal® Clear is a light-curing, transparent fissure sealant.

Composition

Helioseal Clear consists of Bis-GMA and triethylene glycol dimethacrylate (> 99 wt%). Additional contents are stabilizers and catalysts (<1 wt%).

Indication

Helioseal Clear is suitable for the sealing of pits, fissures and foramina caeca.

Contraindication

- If the patient is known to be allergic to any of the material's ingredients.
- If a dry working field cannot be established.

Side effects

In individual cases, contact allergies may occur. According to today's standard of knowledge, systemic side effects are not known.

Interactions

None known to date

Preparatory steps

1. Remove the syringe cap by twisting it until it snaps and then pulling it off the syringe.
2. Attach the applicator tip (Fig. 1) and twist it until it is flush with the syringe (Fig. 2).

Fig. 1

Fig. 2: right

Fig. 3: wrong

Step-by-step application procedure

1. Thoroughly clean the enamel surface to be sealed.

2. Isolate the working field, preferably with a rubber dam.

3. Apply an etching gel, e.g. Email Preparator, and let it react for 30 to 60 seconds.

4. Rinse thoroughly.

5. Dry with water- and oil-free air. The etched enamel should have a mat white appearance. Avoid contamination of the etched surface with saliva.

6. Apply Helioseal Clear directly with the disposable cannula or a disposable brush, and disperse.

7. Wait for approx. 15 sec. Then cure the sealant with a suitable polymerization light (e.g. bluephase®) for 20 sec.

8. Check seal and occlusion.

Warning

Avoid contact of unpolymerized material with skin/mucous membrane or eyes. Unpolymerized Helioseal Clear may cause slight irritation and, in rare cases, may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Special note

Helioseal Clear syringes must be used exclusively with the application tips provided. Other types of application tips may suddenly pop off during dispensing.

Storage

- Close Helioseal Clear syringe immediately after use
- Store material at 2–28 °C (36–82 °F)
- Shelf life: see date of expiration
- Do not use the material after the date of expiration
- Syringes should not be disinfected with oxidizing disinfection agents.

Keep out of the reach of children.

For use in dentistry only.

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.